
Poids maximum utilisateur / Max. user weight / Maximumgewicht gebruiker / Peso máximo utilizador / Peso max. Utente/Peso máximo do utilizador / Maximales Benutzergewicht

Poids net / Net weight / Nettogewicht / Peso neto / Peso netto / Peso líquido / Eigengewicht

Ce produit est conforme au règlement (UE) 2017/745 relatif aux dispositifs médicaux.
This product is conform with regulation (EU) 2017/745 on medical devices.
Dit product beantwoordt aan Richtlijn (EU) 2017/745 over medische apparatuur.
Este producto es conforme con la Directiva (EU) 2017/745 relativa a los productos médicos.
Questo prodotto è in conformità con la Direttiva (EU) 2017/745 relativa ai dispositivi medici.
Este produto está conforme à Directiva (EU) 2017/745 relativa aos dispositivos médicos.
Dieses Erzeugnis entspricht der Richtlinie (EU) 2017/745 über Medizinprodukte.

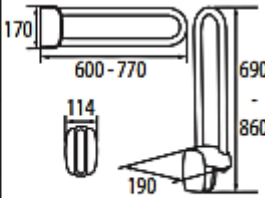Référence G / Reference G / Referentie G / Referencia G / Codice G / Referência G / Referenz G		488601			
Référence P / Reference P / Referentie P / Referencia P / Codice P / Referência P / Referenz P		048860 048865	048880 048885		
Fonction Function Functie Función Funzione Função Funktion	Permet à l'utilisateur de prendre appui en toute sécurité. Allows the user to lean safely. Zo kan de gebruiker in alle veiligheid ondersteund worden. Permite al usuario apoyarse con seguridad. Permette di prendere appoggio con sicurezza. Permite ao utilizador apoiar-se em toda a segurança. Zum sicheren Abstützen des Benutzers.				
Descriptif Description Beschrijving Descrizione Descrição Beschreibung			048880 048885	120 kg	
			048860 048865 488601	150 kg	
			048860 048865 488601	1,81 kg	
		048880 048885	2,04 kg 2,02 kg		
		Numéro notice / Instruction sheet No. / Fichenummer / Número del manual / Numero manuale / Número do manual / Gebrauchsanweisungsnummer		A48860825	
		Date de création / Creation date / Aanmaakdatum / Fecha de creación / Data di creazione / Data de criação / Erstellungsdatum		08/04/2015	
Révision n° / Revision No. / Revisie nr. / Revisión nº / Revisione nº / Revisão nº / Revisionsnr.		04			
Date / Date / Datum / Fecha / Data / Data / Datum		07/09/2023			

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GB **Do not install this equipment before reading the following instructions:**
⚠ Fitting instructions: Advised height for the upper bar: 80cm in toilets and showers, or at the height of the washbasin. Before drilling into the wall, physically hold the product against the wall and mark the position of the holes to be drilled. On walls unable to support sufficient weight, there are 3 possible solutions: use of the vertical support post, the stand, or a reinforcement plate fitted behind the wall. Screws and plugs should be selected according to the type of wall. The 4 fixing points have to be necessarily respected.
⚠ Care and maintenance. IMPORTANT: In order to guarantee the lasting quality of this product and its safety for the user, we recommend that when cleaning it you should use mild soap solutions. **CAUTION: DO NOT USE** cleaning products made of: etones, hydrocarbons, acids, bases, esters and ethers. After use of a cleaning product, make a rinsing in the clear water. **⚠ Safety warnings:** Grab bars add security to the user if they are fitted according to instructions. After fitting and before using, make sure that the fixing screws are tightened securely. Grab bars are not designed to support the total weight of the user; use it for assistance **ONLY**. To avoid any pinching risks, do not put fingers in the mobile parts. The product does not contain any hazardous substances. The manufacturer shall have no liability for any injury caused by improper installation or use of this grab bar. **⚠** At the end of its life cycle, the user must have the product recycled by a certified facility in the country of use.

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