
Poids maximum utilisateur / Max. user weight / Maximumgewicht gebruiker / Peso máximo utilizador / Peso max. Utente/Peso máximo do utilizador / Maximales Benutzergewicht / Макс. вес пользователя.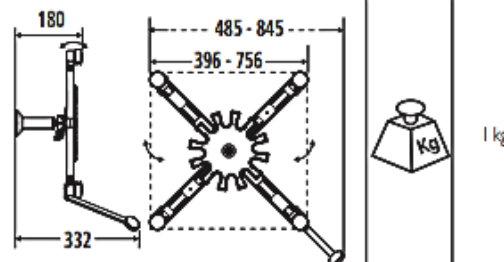

Poids du produit / Product's weight / Productgewicht / Peso del producto / Peso del prodotto / Peso do produto / Gewicht des Produktes / Вес изделия

Référence P / Reference P / Referentie P / Referencia P / Codice P / Referência P / Referenz P / Код P :		047590
Fonction	Permet d'orienter un miroir dans toutes les directions	
Function	Allows to direct a mirror in all the directions	
Functie	Maakt het mogelijk om een spiegel in alle richtingen te oriënteren	
Função	Permite orientar um espelho em todas as direções	
Funzione	Permette di orientare uno specchio in tutte le direzioni	
Função	Permite de orientar um espelho todas as nas direcções	
Funktion	Ermöglicht es, einen Spiegel in alle Richtungen zu schwenken	
Назначение	Позволяет ориентировать зеркало в любом направлении	
Descriptif		
Description		
Beschrijving		
Descripción		
Descrição		
Descrição		
Описание		
Numéro notice / Instruction sheet No. / Fichenummer / Número del manual / Numero manuale / Número do manual / Gebrauchsanweisungsnummer / Номер инструкции	S04759001	
Date de création / Creation date / Aanmaakdatum / Fecha de creación / Data di creazione / Data de criação / Erstellungsdatum / Дата создания	17/07/2008	
Révision n° / Revision No. / Revisie nr. / Revisión n° / Revisione n° / Revisão n° / Revisionsnr. / № редакции	07	
Date / Date / Datum / Fecha / Data / Data / Datum / Дата	09/07/2021	

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GB **Do not install this equipment before reading the following instructions:**

Fitting instructions. Recommended height for the lower part of mirror: 1.05 m. Before drilling into the wall, physically hold the product against the wall and mark the position of the holes to be drilled. Plugs have to be selected according to the type of wall. The 3 fixing points have to be necessarily respected.

Care and maintenance. IMPORTANT: In order to guarantee the lasting quality of this product and its safety for the user, we recommend that when cleaning it you should use mild soap solutions. **CAUTION: DO NOT USE** cleaning products made of: etones, hydrocarbons, acids, bases, esters and ethers. After use of a cleaning product, make a rinsing in the clear water. **⚠️ Safety warnings.** The mirror's support functions correctly if it is fitted according to instructions. It accepts square mirrors from 400 x 400 mm to 700 x 700 mm, rectangular ones, diagonals from 500 to 1000 mm (length inferior or equal twice as big as the width), round ones from Ø 500 mm to 1000 mm, thickness from 4 to 6 mm. Simple adjusting system in position by notched disc. After fitting and before using, make sure that the fixing screws together with mirror's fixings are tightened securely. The handle has to be used only to orientate and tilt the mirror but in any case as a grab bar. To avoid any pinching risks, do not put fingers in the mobile parts. When cleaning hold the mirror during all the process. The product does not contain any hazardous substances. The manufacturer shall have no liability for any injury caused by improper installation or use of this mirror's support. **⚠️** At the end of its life cycle, the user must have the product recycled by a certified facility in the country of use.

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